

Patient: 55-year-old female presented to emergency room with elevated white count and acute necrotizing gangrene to left lateral foot with osteomyelitis.

Past medical history:

- Uncontrolled diabetes, neuropathy

Previous treatment:

- 4th and 5th ray resection with incision and drainage of abscess.
- Systemic antibiotic for 14 days.
- Negative pressure wound therapy (NPWT) for one week post-surgery.
- Bilayer matrix wound dressing was applied for 21 days.

Wound Treatment:

Application of hydrogel and Endoform dermal template, covered by a non-adherent contact layer and Hydrofera Blue classic foam, and secured with roll gauze. Dressing changed weekly.



Week 5:

Wound measurements:

- 1.0 cm x 0.5 cm x 0.2 cm
- 3.0 cm x 1.5 cm x 0.2 cm

Wound Description:

Wound separated into 2 wounds.



Week 11:

Wound measurement:

- 1.0 cm x 0.5 cm x 0.2 cm



Initial wound measurement:

- 9.0 cm x 3.5 cm x 0.3 cm



Week 2:

Wound measurement:

- 8.0 cm x 3.0 cm x 0.3 cm



Week 13:

Wound healed

Case provided by:
Dr. Igor Zilberman DPM, South Florida Lower Extremity Center 2699 Stirling Road Hollywood, FL 33312.

Rx Caution: Federal (USA) law restricts this device for sale or on the order of a physician or licensed healthcare professional. Refer to Instruction for Use for contraindications, warnings, precautions and possible complications.



Manufactured for
Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048
1.888.740.8999

www.hollisterwoundcare.com

The Hollister Logo is a trademark of Hollister Incorporated.
Endoform is a trademark of Mesynthes Limited.
©2014 Hollister Incorporated.

922632-0514